

KEYBOARD SHORTCUT REFERENCE

Radiologist Workflow :

F2	Starts Group Dictation Workflow
F8	When viewing studies from an Unread Folder. This will mark the open study Dictated, and display the next Unread Study on the list. It can also be used to start the dictation workflow with Barcodes.
F9	When viewing studies from an Unread Folder. This will skip the open study without marking it Dictated, and Go to the next Unread Study on the list.
F12	This can be used to mark studies Dictated from the Unread List. This can also be used to mark multiple studies
Alt + D	Sends mark as "Dictate" message to the Dictation system. (Dictation System integration is required.)

Conference Workflow

F8	When viewing items having Ready status from a Conference Folder, this will mark the open item Completed, and display the next Ready item from the Conference Folder.
F9	When viewing items having Ready status from a Conference Folder, this will skip the open item without marking it Completed, and Go to the next Ready item from the Conference Folder.

Auto Compare & Reading Protocol

Alt + C	To Auto Compare the first/next available historical study and go through the list of comparison studies.

Alt + X	Closes all opened Comparison studies
Alt + S	Shows Next Reading protocol Step
Alt + A	Shows Previous Reading Protocol Step

Accessing Patient and Study Information

Alt + I	Displays the study Information
Alt + N	Displays the study Notes
Alt + R	Displays the study Report
Alt + U	Displays Documents in PowerJacket
Alt + E	Displays Series Picker in PowerJacket

Image Manipulation Tool

Both mouse button down then toggle right mouse button	Activates image tool window/level, zoom and pan. Toggle right button to change the tool.
Alt + Left + Mouse move	Right and Left : Window width Forward and Back : Window level
Alt + Ctl + Mouse move	Image zoom or interactive ROI zoom
Alt + Shift + Mouse move	Image pan or interactive ROI size change
O + Left Mouse on Image	Interactive ROI
NumPad + 0 ~ 9	The number pad keys are linked to Preset.
L + Left Mouse button	Lock Tool activate
Esc	Dismiss Locked Image Tool
" = " + Left Mouse button	Zoom to 1X

Viewing Window Control

Ctrl + Mouse move	Changes tile format. (Up-count Change)
F5	Refreshes the window applied F5 key. (Note: It only refreshes the window applied F5. If F5 is applied on the PowerJacket window, only PowerJacket will be refreshed and the refresh behavior may not be as a user intended.) If F5 is applied on the main IE window, then the entire Study/Screen will be refreshed.
F11	Switches between full screen image, "Theatre Mode", and standard windows display with toolbars and navigational items displayed.
Alt + F4	Closes the Active window.

Measurements

R + Left Mouse on Image	Activates Ruler
T + Left Mouse on Image	Activates Text
A + Left Mouse on Image	Activates Arrow
D + Left Mouse on Image	Activates Density Value
G + Left Mouse on Image	Activates Angle
F + Left Mouse on Image	Activates Freehand ROI
E + Left Mouse on	Activates Elliptical ROI

Image	
B + Left Mouse on Image	Activates BookMark

Selecting Images

Ctrl + A	Selects all images or series for image manipulation
Ctrl + Left	Selects multiple images
Shift + Left	Selects sequential image
Left outside of object	De-selects all

Image Navigation

N + Left Mouse on Image	Activates Cine
Home	Returns to first image in the Series/Study
End	Jumps to the last image in the Series/Study
Middle Mouse button down + Mouse move	Displays scrolling images on the selected window. (Stack-in-place)
Page down	Scrolls down one page
Page up	Scrolls up one page
Up arrow	Scrolls up one row of tiles
Down arrow	Scrolls down one row of tiles
Right arrow	Scrolls up one tile
Left arrow	Scrolls down one tile

[IntelliLink and IntelliScroll](#)

I + Left Mouse on	Activates/Deactivates IntelliLink
-------------------	-----------------------------------

Image	
Double clicking mouse wheel	Activates/Deactivates IntelliLink
S + Left Mouse on Image	Activates/Deactivates IntelliScroll
K + Left Mouse on Image	IntelliScroll Add/Remove
C + Left Mouse on Image	IntelliScroll Remove All
J + Left Mouse on Image	Activates/Deactivates IntelliScroll Auto Link

Reference Line

Q + Left Mouse on Image	Lock the reference lines as currently displayed
U + Left Mouse on Image	Unlock reference lines